


Cardinal Health combines radiopharmaceutical delivery elegance with uncompromised radiation safety

Piglet™ & Piglet₂™



1 | At your Cardinal Health pharmacy, a plastic vial and activated charcoal ring are placed into the base of the Piglet™.

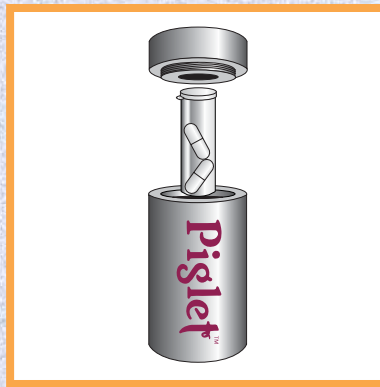


2 | Your patient specific dose (capsules or solution) is compounded and placed inside the Piglet™.



3 | At your nuclear medicine facility, the patient specific dose is carried to the patient administration site inside the Piglet™.



4 | At the administration site, the Piglet™ is opened (snap cap removed) and the dose is administered to the patient. If solution, the septum is penetrated with the radioiodine administration straw, keeping the dose in a closed container at all times.

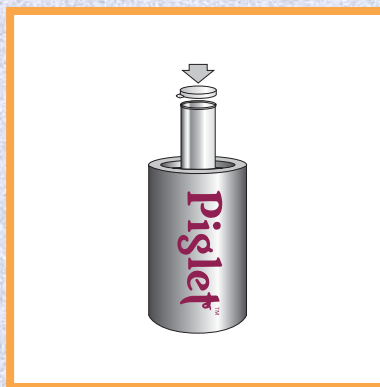

5 | The snap cap is replaced, and the lid reattached to the base of the Piglet™, ready for shipment to your Cardinal Health pharmacy as radiopharmaceutical waste.



6 | At your Cardinal Health pharmacy, the entire snap cap vial is discarded according to the appropriate regulations.